# SPINALOGIC<sup>®</sup>

## Bone Growth Stimulator

Please read before using device

Caution: Federal law (U.S.A. and Canada) restricts this device to sale, distribution or use by or on the order of a physician.

English.....	1
Español .....	19
Deutsch.....	39
Italiano .....	59
Français.....	79
Nederlands ...	99
Ελληνικά ....	119

# Table of Contents

Dear Patient . . . . .	2
Purpose of the Device . . . . .	2
Description of the Device . . . . .	3
When the Device Should Not be Used . . . . .	3
Risks and Benefits . . . . .	3
General Precautions . . . . .	4
Directions for Use . . . . .	5
Putting on the Device . . . . .	5
Starting Each Day's Treatment . . . . .	5
Display Screen . . . . .	7
Checking the Treatment Record . . . . .	7
Safe Battery Care Information . . . . .	8
Replacing the Batteries . . . . .	9
Additional Information . . . . .	10
Device Care and Storage . . . . .	10
Use While Traveling . . . . .	11
Troubleshooting Information . . . . .	12
User Assistance Information . . . . .	14
Customer Care Telephone Numbers . . . . .	14
Glossary of Medical Terms . . . . .	15
Symbols . . . . .	16
Product Specifications . . . . .	17

## Dear Patient,

Do not use this product before you read this manual very carefully. If you have questions, please call your doctor or Customer Care. See page 14 for the Customer Care telephone number in your area.

Words that are written in italics and look like this, “Words”, are explained in the “Glossary of Medical Terms” on page 15.

## Purpose of the Device

Your doctor has asked you to use the SpinaLogic® device on your back after your back surgery (primary lumbar spinal fusion surgery of one or two levels) to help your backbone heal. When used properly, this device will produce a magnetic field over the spot of your back surgery. This field is meant to help the bones heal. To treat yourself, you must wear the device on the outside of your body for 30 minutes per day.

Figure 1. SpinaLogic

## Description of the Device

SpinaLogic (Figure 1, above) is very easy to use, comfortable to wear, and safe to use. It is designed to create a magnetic field that stimulates your backbone to heal. You will not feel the magnetic field it produces when using it. The device consists of a control box (A), a treatment coil (B), and a waist belt (C). It has a single “push button” on the control box to start your treatment and will automatically shut off after 30 minutes. Each day you will hear one beep when you start treatment and two beeps when your treatment is complete. If you hear beeps before your 30-minute treatment is finished, a picture will be shown on the display screen. Refer to the Troubleshooting Information section on page 12 for help. The device also records the number of days that it has been used, which will appear on the display screen. This feature allows you and your doctor to keep track of your care.

## When the Device Should Not be Used

- Do not use SpinaLogic® if you have a heart pacemaker or defibrillator.
- Do not use SpinaLogic if you are pregnant or if you become pregnant.

## Risks and Benefits

- SpinaLogic was not studied on patients who also have metal implants at the spot of their back surgery. Metal implants should not affect your treatment.
- The safety and benefit of SpinaLogic is not known for people whose bones are still growing (generally 18 years old or less).
- If you have any of the following medical problems, it is not known if the device will work for you: osseous or ligamentous spinal trauma, spondylitis, Paget’s disease, severe osteoporosis, metastatic cancer, renal disease, or uncontrolled diabetes mellitus.

- Animal testing with this device has not found any safety problems. However, the chance of safety problems with long time use of this device in people is not known.
- Do not use SpinalLogic if you are unwilling or unable to follow your doctor's orders or the device instructions.
- The device may not work properly and your treatment may be longer unless you do the following:
  - Always follow your doctor's instructions
  - Always follow your daily treatment schedule
  - Always change the batteries when they need it
  - Always take proper care of the device
- The benefit of SpinalLogic is that you have about a 20% greater chance of your back surgery healing if you use the device every day than if you do not use it.

## General Precautions

- Do not use SpinalLogic® near products that may have strong magnetic fields, such as audio speakers. The device may not work properly around these products.
- Do not use SpinalLogic while smoking or near heat, fire or flammable gases because the device may be damaged.
- Do not use SpinalLogic if there are exposed wires or the device appears damaged.
- Do not modify or repair this device because you may damage it.
- Do not put the device or any of its parts in any liquid.
- Do not drop the device or bend the coils because this may damage it.

## Directions for Use

**CAUTION:** Never use the device if the temperature around you is less than 50°F or over 100°F (10°C to 38°C) or the device may not work properly. The device should be in this temperature range for one hour before treating.

## Putting on the Device

Your doctor or a service representative will:

- Make sure the belt on the device fits you;
- Show you how to place the device correctly over the spot of your back surgery;
- Show you how to use the belt to hold the device in place; and
- Show you how to use the device.

## Starting Each Day's Treatment

- Put the device on as you were shown.
- Use the device as you were shown.

Specific directions for operating the device are also given below:

Figure 2. Putting on the device

1. Place the device around the spot of your back surgery. (Figure 2, above).
2. Close and secure the belt.

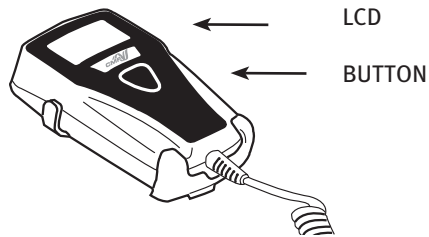
Figure 3. Control Box

3. To start a treatment, press the “push button” next to the liquid crystal display (LCD) screen (Figure 3, above), **hold it down until it beeps**, and then let go.
  - a. The treatment record will be displayed until it beeps.

- b. The 30-minute treatment countdown will begin.

- c. After 30 minutes, the “treatment complete” icon will appear on the LCD screen, the device will beep twice, and it will automatically shut off.

4. Remove the device and store it until the next day. Please see the Device Care and Storage section on page 10 for instructions on the proper storage of the device.

## Display Screen

When using the device, the LCD screen will show the time remaining for your daily treatment. The screen may look like this:

In this example, there are 29 minutes and 37 seconds remaining in the 30-minute treatment.

## Checking the Treatment Record

You may check your treatment record at any time, except during your daily 30-minute treatment.

To view the treatment record, press the “push button” and then let go before it beeps. The LCD screen will look like this:

The number in the upper left-hand corner is the number of days you have successfully treated. The number in the upper right-hand corner is the number of days since you first used the device.

In this example, it has been 204 days since the device was first used, and 201 treatments have been successfully completed.

If the daily treatment has already been completed for that day, and the “push button” is pressed, the treatment record will be shown followed by the “treatment complete” icon. This is a reminder that your daily treatment has already been completed for that day.

## Safe Battery Care Information

Do not do any of the following:

- Never heat or throw the batteries in a fire.
- Never charge the batteries.
- Never let the battery ends contact each other or other metal. Do not let a piece of metal touch both ends of a battery at the same time.
- Never damage or use damaged batteries.
- Never use batteries other than in their correct position.
- Always keep batteries at room temperature, about 70°F (20°C) in less than 80% relative humidity and out of direct sunlight.
- Use ONLY the batteries supplied in the carrying case with SpinaLogic®.
- The batteries may be replaced only after the display turns off.

Note: Be sure to remove plastic from batteries before replacing them.

## Replacing the Battery

The LCD screen will show the following icon when it is time for new batteries. This picture will be shown at the start of a treatment or during treatment. The device will turn off after this picture is displayed.

**Never change the batteries when the device is running.**  
**Wait until the device stops operating to change the battery.**

The picture in Figure 4, below, shows the correct way to replace the battery. When replacing the battery, you will see the correct way to put them in on the inside of the battery compartment. Use ONLY the batteries supplied with SpinaLogic. They are located in the carrying case.

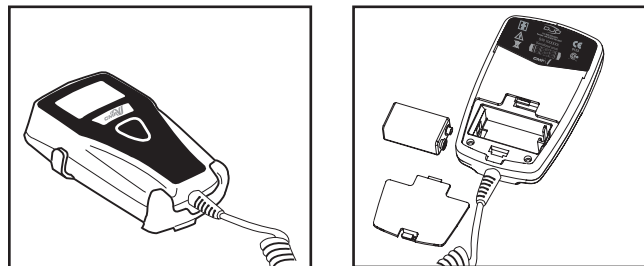
Figure 4. Battery Compartment

## Additional Information

- The device can be used ONLY 270 days in a row (about 9 months). Use the device as long as your doctor tells you to.
- Please take the device with you every time you go to your doctor.
- In a clinical study, SpinaLogic® was tested on 201 people who had the same type of back surgery that you have just had. 67 of the 104 (64%) people who used the device healed. But only 42 of 97 (43%) of the people who did not use the device healed.
- There are alternatives to using SpinaLogic. Some of these are: having physical therapy, taking medications (medicines), seeing a chiropractor, exercising, having surgery, or using other medical devices. Please ask your doctor if you have any questions about other choices for treating your back problem.

## Device Care and Storage

- If needed, clean the surface only with a damp cloth.
- Always store the device in its carrying case in a cool, dry place. Never keep it where the temperature is less than 5°F (-15°C) or greater than 140°F (60°C). Temperatures below or above could damage the device.
- Never store the device or any of its parts in an automobile in cold or hot weather.
- Always store SpinaLogic in the carrying case provided with the device. The carrying case will protect the device when not in use. It also stores the extra batteries and this manual.
- To avoid the risk of choking or strangulation, always store SpinaLogic in its carrying case out of reach of small children.

## Use While Traveling

You should try to use SpinaLogic at the same time every day. The clock inside the device counts each day starting at midnight Pacific Standard Time (PST), 7:00 AM Greenwich Mean Time (GMT), and will only allow one treatment every 24 hours. If you travel outside of your normal time zone, you should try to treat at the same time you normally would at home. Do not take the SpinaLogic device through airport x-ray machines because the x-rays may damage the device.

## Troubleshooting Information

If you hear beeps before your 30-minute treatment is complete, there may be a problem with your device. Look at the LCD screen for a picture. Table 1 below shows pictures that may appear on the LCD screen and other common problems (left side of table). The action to take for each picture or problem is on the right side of the table.

**Table 1. Display Messages**

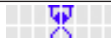
Display Picture	Action to be Taken
	<p>Magnetic Interference – announced by three beeps. The “X” will flash.</p> <ol style="list-style-type: none"> <li>1. If the device looks damaged, call Customer Care.</li> <li>2. If not, try moving to another location in your house. Make sure you are not near metal objects.</li> <li>3. If the picture is still displayed, replace the batteries with new ones.</li> <li>4. If the device still shows the picture, call Customer Care.</li> </ol>
	<p>Replace Battery – announced by three beeps. Replace the battery with a new one.</p>
	<p>Phone For Help – announced by three beeps. Call Customer Care.</p>

(Troubleshooting continues on page 13)

## Troubleshooting Information, continued

Problem	Action to be Taken
The device will not turn on, but there is no error message. You may still hear a beep.	<ol style="list-style-type: none"> <li>1. Try replacing the battery with a new one.</li> <li>2. Make sure to remove plastic off of battery.</li> <li>3. Wait until tomorrow and try treating again.</li> <li>4. If the device still will not turn on, call Customer Care.</li> </ol>
There are lines or bars on the display or the display is too dark to read.	Call Customer Care.
You do not hear any beeps.	<ol style="list-style-type: none"> <li>1. The beeper may not work if you live where it is very humid. The device will still work.</li> <li>2. If you do not live where it is humid, listen again tomorrow and try treating again. Have somebody else listen too.</li> <li>3. If the device still does not beep, call Customer Care.</li> </ol>
The device appears damaged.	Call Customer Care.
Your broken bone hurts more than it used to.	<ol style="list-style-type: none"> <li>1. Make sure that you are treating in a comfortable, relaxed position.</li> <li>2. If you still have more pain than you used to, call your doctor.</li> </ol>
You run out of batteries.	Call Customer Care.

A summary is shown below of other pictures that may appear on the LCD screen.

Display Picture	Definition
	Treatment record.
	Time remaining in 30-minute daily treatment.
	Daily treatment successfully completed – announced by two beeps.
	Please wait.

## User Assistance Information

For servicing of SpinalLogic® or further information concerning the use of the device, please contact Customer Care.

When your treatment is completed, you should take it to your local recycle center where the unit may be taken in as an electric recycle, like a television or computer. You may also contact Customer Care for assistance with device disposal. SpinalLogic is not reusable. The device is for single patient use only. It cannot be re-sold or used on multiple patients.

## Customer Care Telephone Numbers

United States	800.263.6004
Canada	800.263.6004
Europe	+44 1483 459 659

## Glossary of Medical Terms

Metastatic cancer:	cancer that has spread to other parts of your body from where it started
Osseous or ligamentous spinal trauma:	injury to the bones and/or tissue in your back or spine
Paget's disease:	a disease that causes your bones to become more sponge-like
Renal disease:	disease of your kidneys
Severe osteoporosis:	a serious condition where your bones become much more fragile and easily breakable
Spinal fusion surgery:	surgery which attempts to join together bones in your back
Spondylitis:	a condition where you have swelling or irritation of your backbones
Uncontrolled diabetes mellitus:	when you are unable to keep control of your insulin level with either diet or insulin shots

## Symbols

EQUIPMENT SAFETY ELECTRIC SHOCK  
CLASSIFICATION TYPE BF APPLIED PART

CAUTION, CONSULT ACCOMPANYING  
DOCUMENTS

MANUFACTURER

TEMPERATURE LIMITATIONS

ELECTRICAL AND ELECTRONIC EQUIPMENT  
(EUROPE)

## Warranty

It is DJO, LLC policy that all complaints about our devices be communicated to Customer Care at 800-263-6004. The Customer Service Representative will provide technical assistance to determine the extent of the problem or educate the patient on the use of the device. If the device cannot be made to function properly, a replacement device is sent to the patient free of charge and the malfunctioning device is brought back for testing and evaluation. This procedure can be done at any time, and as many times as necessary, during the course of treatment. The course of treatment is defined as the nine (9) month period between the time the device is initiated and the Two Hundredth and Seventieth (270th) treatment.

## Product Specifications

Manufacturer	DJO, LLC 1430 Decision Street Vista, CA 92081-8553 U.S.A.
Model Type	SpinaLogic®
Operating Temperature Range	+50°F (10°C) to +100°F (38°C) Note: CMF Bone Growth Stimulators must remain at Operating Temperature one hour prior to use
Storage and Transportation Temperature Range	+5°F (-15°C) to +140°F (60°C)
Operating Humidity Range	Maximum 85% Relative Humidity
Dimensions	11 in x 12 in x 1.5 in (28 cm x 31 cm x 4 cm)
Weight	2.0 lbs (0.9 kg)
Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide	
Degree of Protection Against Ingress of Liquids	Ordinary Equipment
Degree of Protection Against Electric Shock	TYPE BF Applied Part
Power Supply	Battery Operated, Replaceable, 9 Volts Direct Current
Mode of Operation	Short time continuous operation, 30 minutes per day
Electromagnetic Compatibility (EMC) Electromagnetic Immunity (EMI)	This products is in conformity with Directive 89/336/EEC

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© 2015 DJO, LLC