
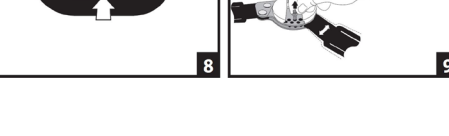
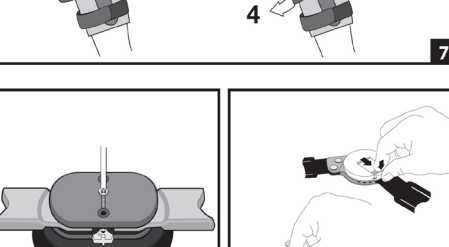
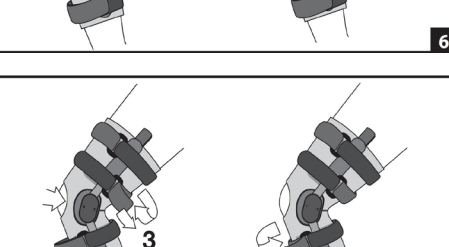
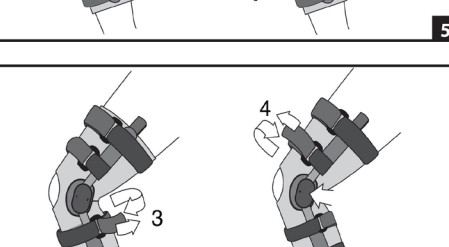
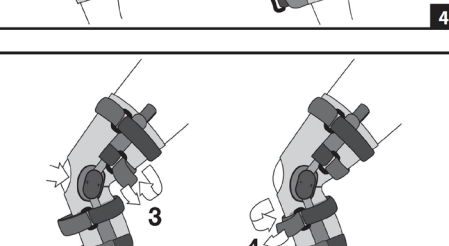
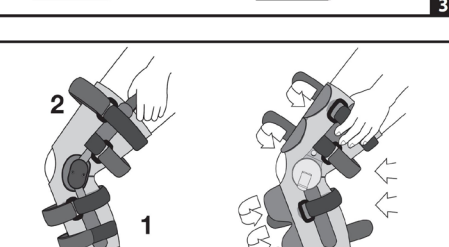
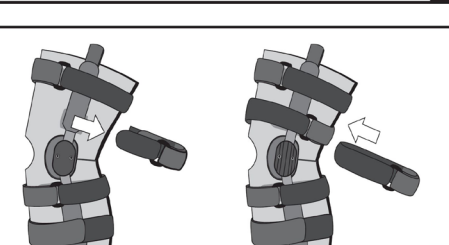
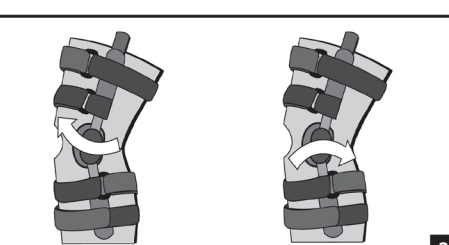
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver... **INTENDED USE/INDICATIONS:** The Donjoy Playmaker Knee Brace is designed to support the knee and provide motion following mild to moderate ACL and/or PCL instabilities... **CONTRAINDICATIONS:** None. **CONVERSION INSTRUCTIONS:** The Playmaker sleeve is shipped in the ACL configuration but may be converted into the ACL or PCL configuration...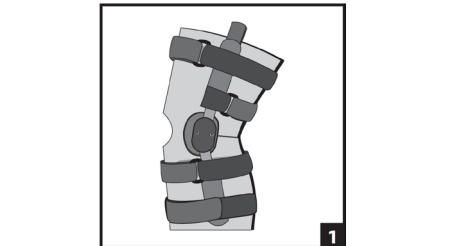

CONVERSION INSTRUCTIONS: The Playmaker sleeve is shipped in the ACL configuration but may be converted into the ACL or PCL configuration... **ACL SLEEVE VERSION AS SHIPPED:** 1) PCL sleeve Conversion - Remove lower high strap and reverse lower D-rings to front side...

APPLICATOR INSTRUCTIONS: A. Fasten and loosen the straps at the thigh and calf. B. Pull the sleeve onto the leg using the loops at the top of each hinge bar. C. Adjust the lower calf strap first (N°1), then the upper calf strap (N°2).

ACL CONFIGURATION SLEEVE VERSION ONLY: 1. Maintaining the position of the hinges, secure the lower thigh strap (N°3). 2. Maintaining the position of the hinges, secure the upper calf strap (N°2).

CI CONFIGURATION SLEEVE VERSION ONLY: 1. Maintaining the position of the hinges, secure the lower thigh strap (N°3) and then the upper calf strap (N°4) on the back of the leg.

ROM HIINGE: Press down, slide out, and lift up the hinge security latch. Rotate the cover until the desired flexion or extension pin is exposed. **ROM POLYCENTRIC HIINGE:** Press down, slide out, and lift up the hinge security latch. Rotate the cover until the desired flexion or extension pin is exposed.

WARRANTS AND PRECAUTIONS: ROM settings should not be changed without supervision of a medical professional. This product must be prescribed and fitted by a healthcare professional. **WARNINGS:** ROM settings should not be changed without supervision of a medical professional.

ONLY: NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

FOR SINGLE PATIENT USE ONLY: **NOT MADE WITH NATURAL RUBBER LATEX.**

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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