

Colles Splint

PATIENT APPLICATION



BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:

- Forearm splint: Immobilization of the wrist and forearm in the initial short and or most severe stage of injury.
- Metacarpal Splint & Mason Allen Splint: Immobilization of the hand and wrist in the initial short and or most severe stage of injury.
- **Colles Splint / Cock-up Splint:** Immobilization of the wrist in the initial short and or most severe stage of injury.
- Elbow Splint: Immobilization of the elbow in acute stage.

A) Place splint on the hand/wrist/forearm/elbow in the desired position and secure with an elastic wrap. Wrapping outside of arm/wrist to inside of arm/wrist.

USE AND CARE:

- The splints may be wiped down with a damp cloth.
- Splints are malleable and may be contoured for individual requirements.

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

WARNING:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.



DJO, LLC

T 800.336.6569 D 760.727.1280 F 800.936.6569

1430 Decision Street | Vista, CA 92081-8553 | U.S.A.

DJOglobal.com