PROCARE®

Deluxe Shoulder Immobilizer

Patient Instructions

APPLICATION & CARE INSTRUCTIONS:

BEFORE USING THIS MEDICAL DEVICE, PLEASE READ THE **INSTRUCTIONS FOR USE** COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:

For immobilization of the shoulder following surgery, injury or cast removal.

WARNING:

THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

APPLICATION INFORMATION:

- Wrap and secure the chestband around the rib cage.
- Wrap and secure the upper arm strap.
- Wrap and secure the wrist strap.

For immobilizers with shoulder straps:

Bring strap over uninvolved shoulder and secure.

For immobilizers with detachable straps:

Secure straps to chest band prior to application of brace.

USE AND CARE:

Hand wash in warm water with mild detergent. Rinse thoroughly and air dry.

WARRANTY:


DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY

NOT MADE WITH NATURAL RUBBER LATEX

For product assistance, please contact Product Support at 1.888.405.3251 or email product.specialist@djoqlobal.com

Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this poster do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

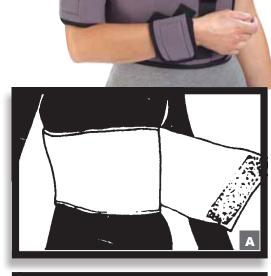







T 800.336.6569 D 760.727.1280 F 800.936.6569 2900 Lake Vista Drive | Dallas, TX 75067 | U.S.A.

D IOGIobal com