Quick Start Guide

DonJoy® IceMan® CLEAR3 Cold Therapy Unit with Universal Wrap-On Pad

- THIS DEVICE CAN BE COLD ENOUGH TO CAUSE SERIOUS INJURY.
- DO NOT use this device without a prescription from a physician.
- READ AND UNDERSTAND ALL WARNINGS AND INSTRUCTIONS FOR USE BEFORE USING THIS DEVICE. ADDITIONAL WARNINGS APPEAR IN THE INSTRUCTIONS FOR USE.





Directions for use of the IceMan® CLEAR


Add ice to fill line.



Add cold water to fill line.



Place lid on device making sure the label is facing up. Secure the lid by raising the handle, which will enage the lid locking mechanism.

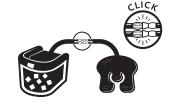

Connect cold therapy hose to cooling pad. To ensure a reliable connection, "snap" or "click" hoses together into place so that the fit is tight and snug.



To turn device on, insert cord into connection on the back of the device and plug power suppply into the wall outlet.

Directions for use of the Wrap-On Pad



Place barrier on knee then place wrap-on pad on knee with tube up as shown. Place round flap on top of patella.



Take strap and wrap around knee from one side of pad over the patella. Wrap should be snug but not tight.



Secure long strap to itself to keep everything flat.

- Apply cold pad and barrier to affected area, making sure to use your prescription.
- WARNING! When applying the cold pad, DO NOT let any part of the cold pad touch your skin.
- Always use with a barrier between your skin and the cold pad.
- Check for moisture on the barrier between your skin and cold pad.
- If moisture is present on the barrier, immediately discontinue use of this device.
- Do not use this device without a prescription from a physician.

Product Support | +1.888.405.3251

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.



T 800.336.6569 D 7607271280 F 800.936.6569 1430 Decision Street | Vista, CA 92081-8553 | U.S.A. DJOGlobal.com/donjoy



SHOULDER

Quick Start Guide

DonJoy® IceMan® CLEAR3 Cold Therapy Unit with Universal Wrap-On Pad

- THIS DEVICE CAN BE COLD ENOUGH TO CAUSE SERIOUS INJURY.
- DO NOT use this device without a prescription from a physician.
- READ AND UNDERSTAND ALL WARNINGS AND INSTRUCTIONS FOR USE BEFORE USING THIS DEVICE. ADDITIONAL WARNINGS APPEAR IN THE INSTRUCTIONS FOR USE.





Directions for use of the IceMan® CLEAR

D

Directions for use of the Wrap-On Pad



Add ice to fill line.



Place barrier on shoulder. Position wrap-on pad on shoulder with tube down. Place round flap on core of shoulder.



Add cold water to fill line.



Take short strap and wrap from one side of pad under the arm. Wrap should be snug but not tight.

3



Place lid on device making sure the label is facing up. Secure the lid by raising the handle, which will enage the lid locking mechanism.



Fold flaps in towards the center

4



Connect cold therapy hose to cooling pad. To ensure a reliable connection, "snap" or "click" hoses together into place so that the fit is tight and snug.



Wrap long strap under uninvolved arm and across torso as shown.

5



To turn device on, insert cord into connection on the back of the device and plug power suppply into the wall outlet

E

Secure long strap to itself to keep everything flat.

- Apply cold pad and barrier to affected area, making sure to use your prescription.
- WARNING! When applying the cold pad, DO NOT let any part of the cold pad touch your skin.
- Always use with a barrier between your skin and the cold pad.
- Check for moisture on the barrier between your skin and cold pad.
- If moisture is present on the barrier, immediately discontinue use of this device.
- Do not use this device without a prescription from a physician.

Product Support | +1.888.405.3251

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.



T 800.336.6569 D 7607271280 F 800.936.6569 1430 Decision Street | Vista, CA 92081-8553 | U.S.A. DJOGlobal.com/donjoy